

QUADRASTEP SYSTEM®

THANK YOU For purchasing our
QUADRASTEP® Starter Kit

Now let's help you get started!
Here is how you set up and use your Fit Kit:



HOW IS THE KIT SET UP?

Your **QUADRASTEP SYSTEM®** Fit Kit has 6 drawers, one for each size orthotic, and one for extra materials. You will see a label on each drawer indicating the size of the orthotics that should be stored in that drawer.

In the bottom drawer you will find your marketing materials in a portfolio, along with your waiting room brochure holder filled with **QUADRASTEP®** patient's brochures. These materials are enough to get you started, additional copies can be downloaded to print on our website nolaro24.com/kit.html. Look for the "marketing material" link.

WHAT WILL I FIND IN EACH DRAWER?

Your **QUADRASTEP SYSTEM®** Fit Kit has 6 drawers, one for each size **QUADRASTEP®** orthotic.

In the first drawer you will find Size 0 if the **E+ Orthotic (Dark Red)**, and size 1 of all of the orthotics, one each **QUAD A (Yellow)**, **QUAD B (Purple)**, **QUAD C (Blue)**, **QUAD D (Green)**, **QUAD E (Red)**, **E+ (Dark Red)** and **QUAD F (Orange)**. Drawer 2 will have all of the Quads (colors) in Size 2, and so on through the 5 drawers/5 sizes. In the 6th drawer you will find 2 pairs of orthotics, Quads **C** and **D** in size 6, as well as your marketing materials and brochure holder.

It is best to keep your orthotics organized in this way for easy access when fitting a patient. That way you will always know what is available and have it handy. Another trick is to always arrange your drawers so that the most commonly dispensed orthotics are in the front, and the least dispensed in the back, as shown.



SHOULD I SELL OUT OF MY KIT?

Your **QUADRASTEP SYSTEM®** Kit has one pair of orthotics in each size and each Quad, for a total of 38 pairs. We recommend that you keep your Kit available for fitting, then order a new pair of **QUADRASTEP®** orthotics to dispense to your patient. The orthotics in your Kit can be disinfected with any household disinfectant. You can also purchase a **QUADRASTEP SYSTEM®** Fit Kit for dispensing.

Congratulations on your first purchase!

QUADRASTEP
SYSTEM®
Prefabricated Custom To Foot Type Orthotics



littleSTEPS®
foot orthotics for kids

You are just 6 simple **STEPS** away from a successful orthotics program!

STEP 1

Schedule your introductory training webinar with your **Nolaro24** sales representative to make sure you and your staff know how to properly select a foot type and an orthotic, and how to use your kit as a fitting tool.

STEP 2

Set up your brochure holders to make your customers aware of the benefits of using a **QUADRASTEP®** and/or **littleSTEPS®** orthotic. Display Signs (available on nolaro24.com) and posters throughout your office.

STEP 3

Add **Nolaro24** product graphics to your website to let your customers know that they can purchase **QUADRASTEP®** and/or **littleSTEPS®** orthotics from you. Include a link to our **Nolaro24** patient website so that they can get more in-depth information on the benefits of their orthotics.

STEP 4

Distribute Referral Sheets to your Referral Sources to inform them you are now dispensing **QUADRASTEP®** and/or **littleSTEPS®**. **Nolaro24** will customize these forms with your practice contact information on request.

STEP 5

Consider setting up an on-site foot screening clinic for **QUADRASTEP®** and/or **littleSTEPS®** foot orthotics to jump start your program. See our Clinic Solutions Pack on nolaro24.com for free guidelines and marketing materials.

STEP 6

Schedule a visit or webinar with your sales representative approximately 6 months after your initial purchase for a refresher your **Nolaro24** products and to develop a strategy for reorder sales to your customers.



Don't forget, we are here to help you every **STEP** of the way!

NOLARO24™
LLC
the alternative to custom orthotics

80 Turnpike Drive, Unit 2B • Middlebury, CT 06762
877.792.4669 • nolaro24.com • info@nolaro24.com

QUADRASTEP SYSTEM®



THANK YOU

For joining our **QUADRASTEP®** Family!

Here are answers to some commonly asked questions

HOW DO I RE-ORDER?

We have enclosed an order form for your convenience. You may fill it out and fax it to the number on the form, or call us to re-order.

HOW DO I EDUCATE MY STAFF?

At anytime you may contact Roberta Nole, our Director of Education. She can set up a webinar or let you know more about our upcoming courses. Contact Roberta at robertan@nolaro24.com.

HOW DO I KNOW IF I'VE GIVEN MY PATIENT THE CORRECT SIZED FOOT ORTHOTIC?

Fitting instructions are enclosed. For further assistance please feel free to call your sales representative.

CAN QUADRASTEP ORTHOTICS BE MODIFIED?

Yes. You can modify a **QUADRASTEP®** orthotic in any way you would modify or adjust a custom orthosis, with one exception. **QUADRASTEP®** orthoses are composed of a thermoplastic rubber and cannot be heat adjusted. If you would like us to assist you with modifications or have a topcover added, we can do this for an additional fee.

WHAT KIND OF INSTRUCTIONS DO I GIVE MY PATIENT?

You may duplicate the enclosed patient information card, or you can print information from our website www.thequadrastepssystem.com to hand to your patient.

SHOULD I DISINFECT FOOT ORTHOTICS BETWEEN PATIENTS?

If you are using your orthotics as a fitting tool, please disinfect with any topical disinfectant you are currently using between patients. Once your patient has purchased their orthotic, they can be cleansed with soap and water.

MAY ORTHOTICS BE RETURNED?

Unused product, in its original packaging, may be returned for credit within the first 30 days of invoicing. Shipping will also be credited to your account for any returns due to our error, otherwise shipping is your responsibility.

For more FAQs please see our website nolaro24.com/faq

QUADRASTEP SYSTEM®

The QUADRASTEP SYSTEM® is based on a clinical algorithm* identifying 24 unique foot types. These 24 foot types are subdivided into 6 groupings of 4, known as "QUADS". Each QUAD has its own very specific foot and gait characteristics.

Why Treat all feet the same?

An individual's foot type (quad type) can influence not only how they walk, but the conditions and pathologies that may afflict them throughout their lives. Unlike other pre-fabricated orthoses that have a "one-shape-fits-all" limitation, the QUADRASTEP SYSTEM® offers a more tailored solution for each foot type.

24 FOOT-TYPE ASSESSMENT MODEL

REARFOOT VARUS DEFORMITIES

UNCOMPENSATED VARUS		COMPENSATED VARUS		
LARGE	MEDIUM	SMALL		
#1	#2	#3	#4	LARGE
A		B		
#5	#6	#7	#8	SMALL
#9	#10	#11	#12	NEUTRAL
C		D		FOREFOOT
#13	#14	#15	#16	SMALL
#17	#18	#19	#20	MEDIUM
E		F		FOREFOOT
#21	#22	#23	#24	LARGE
				VARUS DEFORMITIES

*U.S. Patent No. 7,789,840

LEFT FEET SHOWN ABOVE

THE QUADRASTEP SYSTEM®

A SEVERE PES CAVUS

- Poor Shock Attenuation
- Excessive Supination
- Narrow or Cross Over Gait

Propels forcefully from 1st Metatarsal

KEY ORTHOTIC FEATURES

- Lateral Forefoot Posting
- 1st MTH Cut-Out
- Deep Lateral Heel Cup
- Equinus Correction
- Great for Ankle Instability!**

POSSIBLE CLINICAL SYMPTOMS

- Lateral Ankle Instability
- Peroneal Tendonitis
- Heel Pain
- 5th Metatarsal Base Pressure
- Lower Back Pain
- Sesamoiditis, Claw Toes
- Knee Recurvatum

CALLUSES

FOOT PROGRESSION ANGLE

B MILD PES PLANUS

- Pronates through Mid-Stance
- Re-supinates in Propulsion
- Propels off 1st and 2nd MTH's

Toe In Gait

KEY ORTHOTIC FEATURES

- Medial RF Posting
- Intrinsic Lateral FF Posting
- Mild Medial Skive
- Great for In-Toeing!**

POSSIBLE CLINICAL SYMPTOMS

- Toe-In Gait
- Neuromas
- Sesamoiditis
- 1st Ray Hypermobility
- Sacro-Iliac Pain
- Often Unilateral if Associated with Leg Length Inequality

CALLUSES

FOOT PROGRESSION ANGLE

C NEUTRAL FOOT

- Poor Shock Attenuation
- Restricted STJ Pronation
- Propels off Medial Hallux

Toe Out Gait

KEY ORTHOTIC FEATURES

- Neutral RF Posting
- Medium Arch
- Standard Depth
- Great for Hip and Back Pain!**

POSSIBLE CLINICAL SYMPTOMS

- Hip Pain
- Lower Back Pain
- Iliotibial Band Syndrome
- Retrocalcaneal Heel Pain
- Haglund's Deformity
- Medial Hallux Pinch Callus

CALLUSES

FOOT PROGRESSION ANGLE

D MODERATE PES PLANUS

- Neutral Toe Out
- Pronation through Midstance
- Midtarsal Joint Instability

Propels off 2nd and 3rd Metatarsal (due to Transverse Metatarsal Arch Reversal)

KEY ORTHOTIC FEATURES

- Deep Heel Cup
- Medial RF Posting
- Moderate Medial Skive
- Medial Flare
- Great for Plantar Foot Pain!**

POSSIBLE CLINICAL SYMPTOMS

- Plantar Fasciitis
- Metatarsalgia
- Functional Hallux Limitus
- Patellofemoral Pain Syndrome
- Posterior Tibial Tendonitis
- Neuromas
- Dorsal Bunions

CALLUSES

FOOT PROGRESSION ANGLE

E ABDUCTOVARUS FOREFOOT

- Narrow Heel Base Gait
- Restricted Subtalar Pronation
- Pivots at 5th MTH in Propulsion

Medial Heel Whip

KEY ORTHOTIC FEATURES

- Medial RF & FF Posting
- 5th MTH Cut-Out
- 1st MTH Control
- Medial Flare/Flange
- Great for Heel, Shin & Knee pain!**

POSSIBLE CLINICAL SYMPTOMS

- Plantar Fasciitis / Heel Pain
- Shin Splints
- Knee Pain
- Tailor's Bunions
- Calcaneal Apophysitis (Sever's Disease)
- Osgood-Schlatter Disease

CALLUSES

FOOT PROGRESSION ANGLE

F SEVERE PES PLANOVALGUS

- Pronates through Propulsion
- Severe MTJ Instability
- Propels from Central MTH's

Lateral Column Instability

KEY ORTHOTIC FEATURES

- Depth Orthosis
- Large Medial Skive
- Medial RF & FF Posting
- 1st MTH Cut-Out to ↑ Peroneal Function
- Great for PTTD!**

POSSIBLE CLINICAL SYMPTOMS

- Posterior Tibial Dysfunction
- Tarsal Tunnel Syndrome
- Plantar Fasciitis
- Knee Valgus/DJD
- Subfibular Impingement
- HAV/Bunions
- Splayfoot

CALLUSES

FOOT PROGRESSION ANGLE

LEFT FEET SHOWN ABOVE

*U.S. Patent No. 7,789,840

877.792.4669

Why Treat All Feet The Same?

info@nolaro.com • nolaro24.com

QUADRASTEP SYSTEM®

is a product of

NOLARO²⁴ LLC
the alternative to custom orthotics

STEP 1

Identify medial arch height

CAVUS MEDIUM TO HIGH

CHOOSE FROM:



PLANUS MEDIUM-LOW TO LOW


CHOOSE FROM:



STEP 2

Identify foot & leg position

CHOOSE FROM

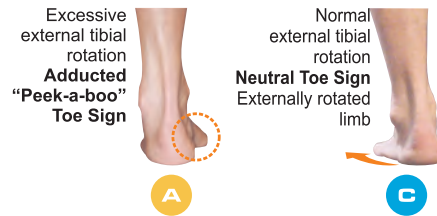

OR CHOOSE FROM



STEP 3

Identify gait style

CHOOSE FROM




A Narrow Gait



C Toe-Out Gait


OR CHOOSE FROM

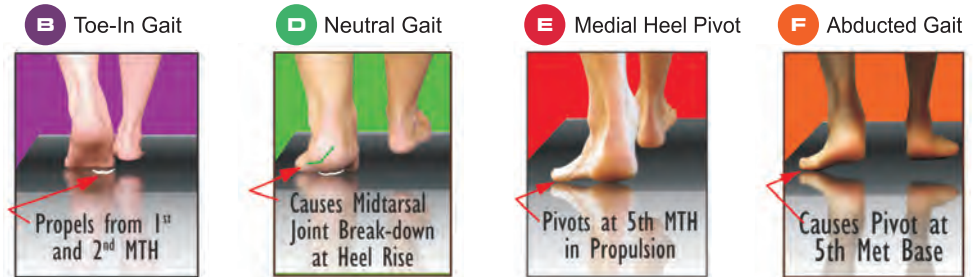

STEP 4

Check against callus pattern

CHOOSE FROM

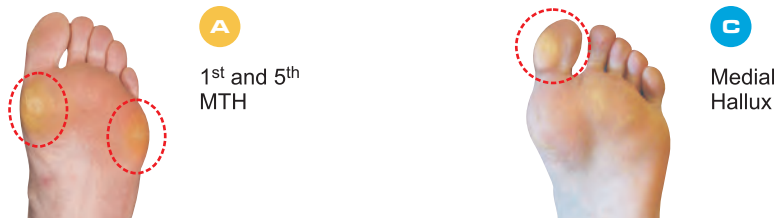

OR CHOOSE FROM

B 1st and 2nd MTH D 2nd MTH E 2nd and 5th MTH F Large Central MTH



4 Easy Steps to Choosing a Foot Type



You can download our Free NEW APP and use the Easy Foot Typing Tool Just scan the QR code with your smartphone or go to: <http://m.magmito.com/121342/326578storet>

Nolaro24™ : Bridging the Gap Between Custom and Non-Custom Orthotics

By Louis DeCaro, DPM

Nolaro24™, LLC, founded by Roberta Nole and myself, have proudly supplied the QuadraStep Orthotic System and littleSTEPS line of pediatric foot orthoses and gait plates to the podiatry profession over the last 5 years. These devices provide today's podiatrist with an entire line of pre-fabricated functional foot orthoses to treat children and adults with an alternative to custom.

In today's practice, the "ideal" method of treating patients via custom orthoses is often time and cost-prohibitive due to declining reimbursements. In contrast, off-the-shelf "arch supports", although less expensive, usually lack adequate biomechanical support. The QuadraStep Orthotic System™ bridges the gap between custom and non-custom orthoses.

The QuadraStep Orthotic System™ is based on Nole's 10+ years of research, refinement and practical application of a foot typing model. It details a specific methodology implementing a clinical algorithm that categorizes a patient's foot into one of 24 types. Trained podiatrists are able to classify a patient's foot type and dispense prescription-based orthoses within minutes.

The QuadraStep Orthotic System™ simplifies the 24 foot-type medical model into 6 subclassifications known as "Quads", groups of four relatively similar foot types. The 6 Quads are lettered A through F, and are color coded for further clarification.

Unlike other off-the-shelf orthotic systems that offer only a "One-Shape-Fits-All" arch support, the QuadraStep Orthotic System™ offers 6 Quad specific functional orthoses, each biomechanically tailored to that of a true custom prescription.

The QuadraStep Orthotic System™ offers several distinct advantages to podiatrists. First, it is easy to learn and can be quickly applied in a clinical setting. Second, it eliminates the need for costly and messy foot impressions that often are prone to error. Third,

there is no wait. The orthoses can be stocked in-house and ready for immediate delivery. Fourth, and most importantly, the product offers truly custom results at only a fraction of the cost of traditional custom orthoses.

The original QuadraStep System was produced with 6 adult size ranges in one full width that provided complete biomechanical control. *While very successful, we recognized that many practitioners wanted a lower volume device with similar biomechanical properties. As a result, we now offer a NARROW version of the QuadraStep Orthoses. The NARROW versions offer the same exact posting prescription and materials as the original width, but with an overall decrease in width by a quarter inch, including a slightly less deep heel cup.*

This narrow version is designed to fit in more casual style shoes such as loafers, and also fits a narrower foot better. The NARROW version is a popular choice for patients in ordering "2nd pairs" for their more casual shoes. Their original pair provides complete mechanical control in their full fitting and athletic shoes and the 2nd NARROW pair gives them flexibility to wear "nicer" shoes with similar biomechanical control.

Today's decline in economic conditions and rising co-pays makes it difficult for some patients to include a foot orthoses

program into a family budget. This results in an increased demand on today's podiatric physician to find ways to provide quality health care. The QuadraStep Orthotic System™ enhances podiatric practice management by simultaneously addressing budgetary concerns and the need for immediate orthotic intervention. In addition, it offers the podiatrist an opportunity to expand their orthotic practice in a timely and efficient manner.

For more information contact Nolaro24 at info@thequadrastepsystem.com, visit www.thequadrastepsystem.com or call 877 792-4669, or circle #155 on the reader service card.

SALES REP _____

CUSTOMER # _____

DATE _____

Submit orders to: Tel **877.792.4669** Fax **(203) 758-1011** E-mail **orders@nolaro24.com**

Office Name: _____ Phone () _____
 Type of Practice: _____ Fax () _____
 Contact Name: _____ PO# _____
 Email Address: _____

Ship To:

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Bill To: (if different)

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Credit card and billing address: (if different) Keep credit card on file for future orders? Y N Invoice _____

Name: _____ Street or P.O. # _____ Zip _____
 Account # _____ Exp _____ CCV* _____ Amex _____ MC _____ Visa _____ Disc _____

*CCV security code is the last 3 digits on the back of your credit card in the signature box

QUADRATEST PAIRS (please enter number of pairs on line)

With bag Without bag (break-in instructions are still included)

A Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____

B Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____

C Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____
 Size 6 _____

D Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____
 Size 6 _____

*Size 6 available in the C and D Quads ONLY

*Size 6 available in the C and D Quads ONLY

E Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____

F Quad Regular Topcovered Narrow Topcovered
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____

E+ Quad Regular Topcovered Narrow Topcovered
 Size 0 _____
 Size 1 _____
 Size 2 _____
 Size 3 _____
 Size 4 _____
 Size 5 _____

*** Narrow orthotics are 1/4" narrower grind for casual shoes and cleats. No extra charge.**

QS Fitchain _____ 38 pr Kit _____
 QS Marketing Kit _____ Narrow 38 pair Kit _____
 Sample Widget Kit _____ Heel Lifts S M L _____
 Large Widget Kit _____ Met Mounds S M L _____
 24 Foot Type Poster _____ Wedge Strip _____
 6 Quad Poster _____

LITTLESTEPS Foot Orthotics Pairs

Pairs	Qty.	Pairs	Qty.
Size 00	_____	Size 00	_____
Size 0	_____	Size 0	_____
Size 1	_____	Size 1	_____
Size 2	_____	Size 2	_____
Size 3	_____	Size 3	_____
Size 4	_____	Size 4	_____
Size 5	_____	Size 5	_____
Size 6	_____	Size 6	_____
Size 7	_____	Size 7	_____
Size 8	_____		
Size 9	_____	NEW LS Poster	_____
LS Fitchain	_____	LS 11 Kit	_____
LS Marketing Kit	_____	GP 9 Kit	_____

SIZE	MODEL	U.S. KIDS	U.S. MEN	U.S. WOMEN	EURO
9	YOUTH/ADULT*	7.5 to 8.5	7.5 to 8.5	9 to 10	41 to 42
8	YOUTH/ADULT*	6 to 7	6 to 7	7.5 to 8.5	39 to 40
7	YOUTH/ADULT*	4.5 to 5.5	4.5 to 5.5	6 to 7	37 to 38
6	CHILDREN	3 to 4			35 to 36
5	CHILDREN	1.5 to 2.5			33 to 34
4	CHILDREN	13 to 1			31 to 32
3	CHILDREN	11.5 to 12.5			29 to 30
2	CHILDREN	10 to 11			27 to 28
1	TODDLER	8.5 to 9.5			25 to 26
0	TODDLER	7 to 8			23 to 24
00	TODDLER	5 to 6.5			21 to 22

*Youth sizes are roughly equivalent to Men's sizes in length.

SIZING	Size #	US Women	US Men	Euro	UK
	Size 0	5.0 - 6.0	3.5 - 4.5	35 - 36	3.0 - 4.0
	Size 1	6.5 - 7.5	5.0 - 6.0	37 - 38	4.5 - 5.5
	Size 2	8.0 - 9.0	6.5 - 7.5	39 - 40	6.0 - 7.0
	Size 3	9.5 - 10.5	8.0 - 9.0	41 - 42	7.5 - 8.5
	Size 4	11.0 - 12.0	9.5 - 10.5	43 - 44	9.0 - 10.0
	Size 5	12.5 - 13.5	11.0 - 12.0	45 - 46	10.5 - 11.5
Size 6	14.0 - 15.5	12.5 - 14.0	47 - 49	12.0 - 13.5	

All sizes are available in Regular or Narrow Width

*Size 0 available in E+ only, Size 6 available in C and D only

SPECIAL INSTRUCTIONS

Return Policy: Items may be returned for future credit unused and undamaged in original packaging within 30 days of purchase with copy of original receipt. 25% re-stocking fee may apply.

SALES REP _____ CUSTOMER # _____ DATE _____

Submit orders to: Tel **877.792.4669** Fax **(203) 758-1011** E-mail **orders@nolaro24.com**

Office Name: _____ Phone () _____
 Type of Practice: _____ Fax () _____
 Contact Name: _____ PO# _____
 Email Address: _____

Ship To:

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Bill To: (if different)

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Credit card and billing address: (if different) Keep credit card on file for future orders? Y N Invoice _____

Name: _____ Street or P.O. # _____ Zip _____
 Account # _____ Exp _____ CCV* _____ MC _____ Visa _____ Amex _____ Disc _____

*CCV security code is the last 3 digits on the back of your credit card in the signature box

38 PAIR STARTER KIT - BEST DEAL!!

Everything you need to start dispensing
QUADRATEST[®] Orthotics to your patients
TODAY!

- 38 pairs (all sizes and Quads)
- 6 Drawer Storage Cart
- Educational Poster
- Marketing Material
- FREE Training Webinar

Includes 19" x 27"
 Educational Poster



DON'T FORGET!

Want to customize your orthotics yourself? Try a **Widget Kit!** Kit includes a variety of self-stick wedges, heel lifts, and met mounds.

Widget Sampler Kit _____ (\$25 ea.) Large Widget Kit _____ (\$99 ea.)

Kits

____ Regular Width Orthotics ____ Narrow Width* Orthotics

* 1/4" narrower grind for casual shoes and cleats. No extra charge.

Return Policy: Items may be returned for future credit (minus shipping charges) unused and undamaged in original packaging within 30 days of purchase with copy of original receipt. 25% re-stocking fee may apply.

SIZING	Size #	US Women	US Men	Euro	UK
	Size 0	5.0 - 6.0	3.5 - 4.5	35 - 36	3.0 - 4.0
	Size 1	6.5 - 7.5	5.0 - 6.0	37 - 38	4.5 - 5.5
	Size 2	8.0 - 9.0	6.5 - 7.5	39 - 40	6.0 - 7.0
	Size 3	9.5 - 10.5	8.0 - 9.0	41 - 42	7.5 - 8.5
	Size 4	11.0 - 12.0	9.5 - 10.5	43 - 44	9.0 - 10.0
	Size 5	12.5 - 13.5	11.0 - 12.0	45 - 46	10.5 - 11.5
Size 6	14.0 - 15.5	12.5 - 14.0	47 - 49	12.0 - 13.5	
All sizes are available in Regular or Narrow Width					

*Size 0 available in E+ only, Size 6 available in C and D only

The **NEXT STEP** in Foot Orthotic Design
QUADRASTEP
 **SYSTEM**[®]
 Prefabricated Custom To Foot Type Orthotics

WIDGET SAMPLER KIT

Now make your own customized adjustments to our **QUADRASTEP**[®] orthotics! Enhance comfort and effectiveness using our Heel Lifts, Met Domes and Wedge Strips. The kit provides an assortment of self sticking adjustment tools for the ultimate in customization. Each component may also be purchased independently, see order form for ordering information.

Each Kit Includes:

- 1 pr. ea. 4mm, 6mm, and 8 mm Heel Lifts
- 1 pr. Met Mounds
- 4 pcs. Wedge Strip

\$25 ea.

Available for purchase individually:


- | | |
|--|------|
| Heel Lifts in 4mm, 6mm, 8mm (5 pr. pack) | \$15 |
| Met Mounds (5 pr. pack) | \$15 |
| Wedge Strip (per 17" piece) | \$ 7 |

CALL TO ORDER NOW
877.792.4669
 nolaro24.com

QUADRASTEP SYSTEM[®] - AFFORDABLE, READY-TO-WEAR, WITH CUSTOM RESULTS!

QUADRASTEP SYSTEM[®]
 is a product of

NOLARO²⁴ **LLC**
 the alternative to custom orthotics

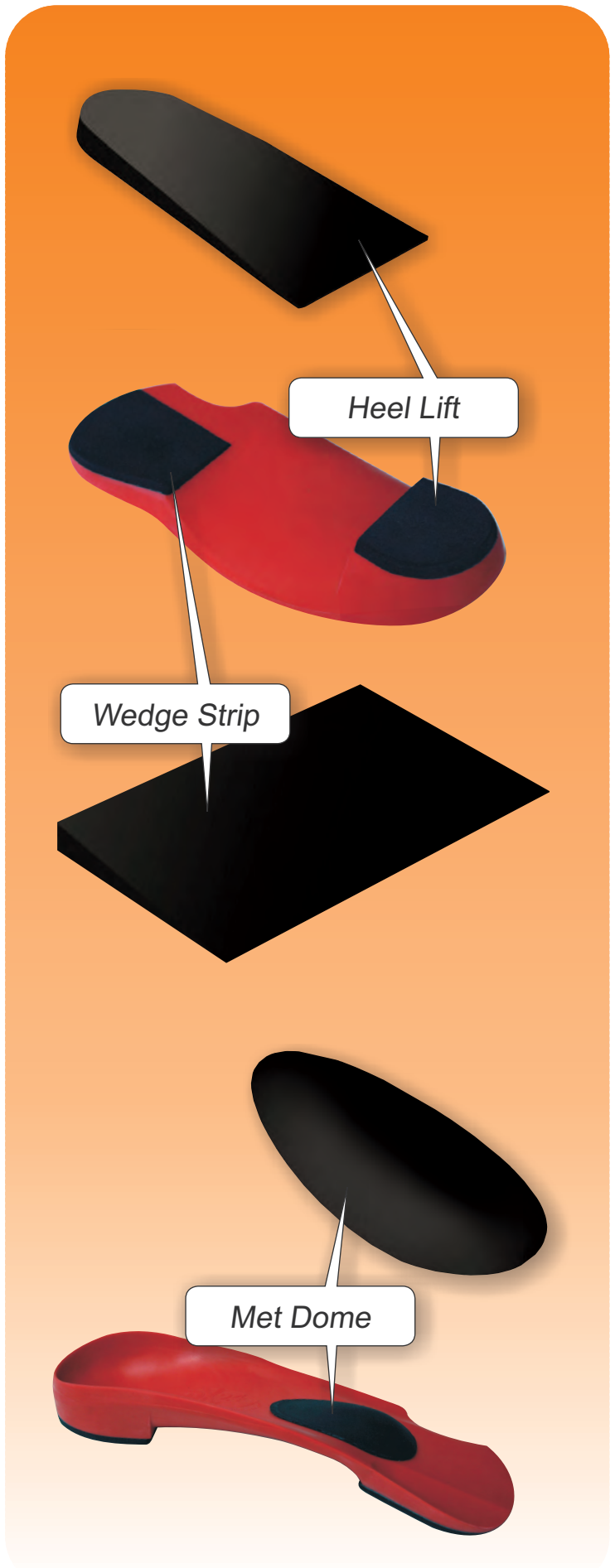

SALES REP _____	CUSTOMER # _____	DATE _____
-----------------	------------------	------------

Submit orders to: Tel **877.792.4669** Fax **(203) 758-1011** E-mail **orders@nolaro24.com**

Office Name: _____ Phone () _____
 Type of Practice: _____ Fax () _____
 Contact Name: _____ PO# _____
 Email Address: _____

Ship To:

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Bill To: (if different)

Name: _____ Address: _____
 City: _____ State _____ Zip _____

Credit card and billing address: (if different) Keep credit card on file for future orders? Y N Invoice _____

Name: _____ Street or P.O. # _____ Zip _____
 Account # _____ Exp _____ CCV* _____ MC _____ Visa _____ Amex _____ Disc _____

*CCV security code is the last 3 digits on the back of your credit card in the signature box

Widgets and Widget Kits

Heel Lifts
\$15.00/pack

4 mm (10 lifts/box)

Size	QTY
Small	_____
Medium	_____
Large	_____

6 mm (10 lifts/box)

Size	QTY
Small	_____
Medium	_____
Large	_____

8 mm (10 lifts/box)

Size	QTY
Small	_____
Medium	_____
Large	_____

Met Domes
\$15.00/pack

(pack of 5 pairs)

Size	QTY
Small	_____
Medium	_____
Large	_____

Wedge Strip
\$7.00 each

QTY _____

Widget Sampler Kit
\$25.00 each

Each Kit Includes:

- 1 pr. Met Domes
- 4 pcs. Wedge Strip
- 1 pr. ea. 4mm, 6mm, and 8mm Heel Lifts


QTY _____

Large Widget Kit
\$99.00 each

Each Kit Includes:

- 8 lg 4mm Heel Lifts 10 pieces Wedge Strip
- 8 lg 6mm Heel Lifts 15 Scaphoid pads
- 8 lg 8mm Heel Lifts 15 Met Domes

QTY _____

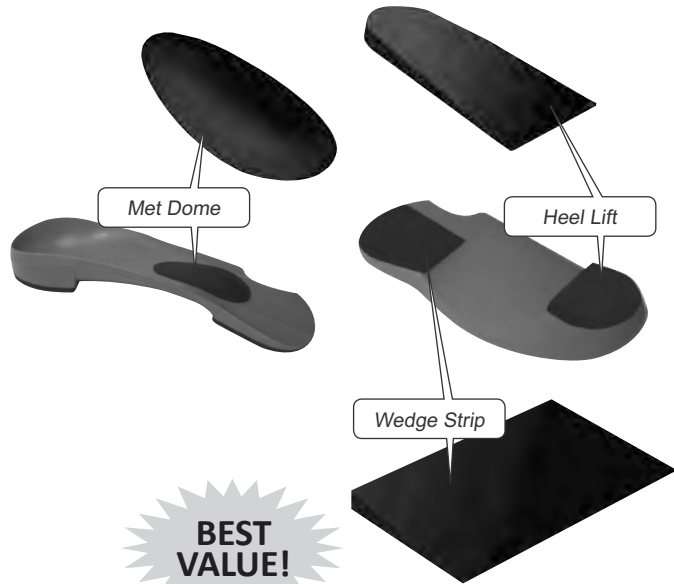

Return Policy: Items may be returned for future credit (minus shipping charges) unused and undamaged in original packaging within 30 days of purchase with copy of original receipt. 25% re-stocking fee may apply.